

## Care and use instructions

Please read before use!

### Indications for use

- For hygienic reasons, DryMed® washable incontinence products must be washed before using.
- Please fold the DryMed® incontinence diapers' hook-and-loop fastener together to avoid damages of their delicate inside caused by the hook-and-loop fasteners.
- For a complete cleaning of DryMed® incontinence products, it is advisable to wash the products inside out.
- Please dry and store DryMed® incontinence products inside out to guarantee the evaporation of any residual humidity.
- DryMed® incontinence products are manufactured in usual ready-to-wear garment sizes.
- While using DryMed® incontinence diaper, please pay attention that hook of fastener do not scratch the skin, which will otherwise cause skin irritations.
- Furthermore, it is important to avoid folds at legs' and hips' surrounding when DryMed® incontinence products are worn. This is to avoid liquid leakage.
- To avoid risk of asphyxiation, please keep the packaging bag away from children.

### Term of use


- After soiling with faeces and/or urine, DryMed® incontinence products should be changed at least after four hours. It is advisable to change the products immediately after using.
- Even without dragging, DryMed® incontinence products should be changed at least after four hours.

### Care instructions

DryMed® incontinence products can be washed with usual detergents in the washing machine. Please do not use fabric softeners otherwise the absorbing volume of product will be reduced.



**Temperature:** Laundering up to 95°C is possible. For ecologic and economic reasons, washing temperature of 60°C is adequate and advisable.

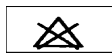

**Whitening:** Please, do not use chlorine; the waterproof PU-covering will be damaged and loses its characteristics (i.e. causes permeability).



**Laundry dryer:** The dryer temperatures are not allowed to exceed 90°C.



**Ironing:** Please, do not iron; the sensible surface will be destroyed by direct heat.

By following written instructions, DryMed® incontinence products can be worn and washed up to 300 times without losing their characteristics.

### Storing Advice

- \* Products should not be stored over 90°C.
- \* Products must not be in contact with open fire.
- \* Please avoid direct contact to other heat sources (heating element support etc).

### Risks

- \* If DryMed®- incontinence products stay in longer contact to body, skin irritations can result.



The manufacturer is not liable for damages or injuries due to improper handling or misuse.