

Anti-mite bed covers

Protection against dust-mite allergy

Care and use instructions 01.2018

Please read before use!

Special use instructions

- For hygienic reasons, WELLline[®] anti-mite bed covers must be washed before using.
- WELLline[®] anti-mite bed covers are used as encasings under usual bed cover.
- After washing, the anti-mite bed covers must be completely dry before using.
- As a security, the inside zip protection must cover properly the zipper.
- There should be no sharp-edged object in contact with the anti-mite bed covers.
- Please pay attention that the right size of anti-mite bed covers is used.
- Anti-mite bed covers must be stored away from children.
- Please, wash anti-mite bed covers with zipper open.
- To avoid risk of asphyxiation, please keep the packaging bags away from children.

Term of use

- Anti-mite bed covers can be washed as often as necessary. By strong allergic reactions, it is advisable to wash the anti-mite bed covers every three months.

Care instructions

Anti-mite bed covers can be washed with usual detergents in the washing machine.



Temperature: For ecologic and economic reasons, washing at 60°C is adequate and advisable



Whitening: chloric whitening is possible



Laundry dryer: dryable in laundry dryer



Ironing: iron at mild temperature

By following the above written instructions, anti-mite bed covers can be used for a longer time.

Storing advice

- Anti-mite bed covers must not be in contact with open fire.
- Please avoid direct contact with other heating sources.



Risks

- By covering anti-mite bed covers overhead there is a danger of suffocation!

Performance characteristics

- High particle retention capacity
- Breathable
- Toxproof through Oeko-Tex Standard 100, class I
- High wash resistance
- High humidity transport
- TÜV certificated



The manufacturer is not liable for damages or injuries due to improper handling or misuse.